



Why LiveLife Alarms?

- Free lifetime support
- Delivered fully programmed and ready to use right out of the box
- No installation required
- 30-day money back guarantee
- 12-month warranty
- NDIS and My Aged Care approved
- Trusted by over 150,000 Australians



How our alarms work



Step 1

Activation

To activate your alarm, press the SOS button. When a fall is detected, the alarm automatically activates.



Step 2

Response - Family and Friends

The alarm sends a text message to your emergency contacts with a Google Maps link showing your location, then it starts the call sequence.

Response - Professional Monitoring

The alarm sends a signal and your GPS location to our 24/7 monitoring centre. Within 60 seconds, a team member will call the alarm to assist you.



Step 3

Help

Whether you are on the family and friends model or the professional monitoring model, you will be able to talk to your elected emergency contact hands-free through the alarm. They can arrange the assistance you require.



More information

For more information or to order your alarm, please visit our website or contact our friendly staff.

www.livelifealarms.com.au
admin@livelifealarms.com.au
Freecall: 1800 936 774



Providing independence, peace of mind and choice!



LiveLife Alarms allow you to live life and stay safe knowing that help is just the press of a button away.



LiveLife Alarms is Australia's largest provider of personal mobile alarms. Since 2016, we've helped Australians stay independent and active with alarms that offer safety and peace of mind for both wearers and their loved ones.

Our mission is to help people live the life they deserve.

LiveLife Pendant

Simple to use, our popular pendant comes with a docking station, lanyard, clothing clip and wristband.



LiveLife Watch

Modern and discreet, our watch comes with a choice of silicone, leather or velcro bands, and feature a variety of customisable watch faces.

Specifications

	Pendant	Watch
Size in millimetres	61 x 42 x 16	47 x 47 x 16
Weight in grams	42	64
Battery	3 to 5 days	Up to 24 hours

Our alarms utilise the Telstra or Optus network, and they will work anywhere in Australia where there is mobile coverage.



Key features



SOS button

When help is needed, press the SOS button to activate your alarm.



Automatic fall detection

When a fall is detected, the alarm will automatically activate and notify your elected emergency contacts.



Two way voice

The built in speakerphone allows crystal clear 2 way voice communication with your elected emergency contacts.



Voice announcements

Voice prompts will guide you through the activation process and are available in seven languages.



GPS location

When activated, the alarm will send your location to your elected contacts via Google Maps. Geofencing options available for those at risk of wandering.



Showerproof

With a waterproof rating of IP7, our alarms are safe to be worn in the shower or rain.



Daily reminders

We can program up to four daily reminders into your alarm.

Additional features (watch only)



Time and date

Can personalise your watch with a variety of digital and analogue watch face options.



Step counter*

Will track your daily steps, distance walked, and calories burned.



Heart rate monitor*

Measures your heart rate in real-time.



Touchscreen

Simple to navigate through your watch features.

Flexible monitoring options

With LiveLife Alarms the choice is yours. With no lock-in contracts, our flexible monitoring service lets you switch models as your situation changes*, ensuring you always have the the right solution to suit your needs.

Family and friends

- Connecting you to your loved ones
- Store up to 6 emergency contacts, including 000
- No monthly monitoring fees, annual renewal applies

Professional monitoring

- Help available 24/7
- Guaranteed response within 60 seconds, at any time of day or night
- No lock-in contracts, monthly monitoring fee applies